biotechne® / ExosomeDx™

The ExoDx™ Prostate Test

At-Home Kit Instructions (English & Spanish)

It's simple!
**Prepare.
Collect.
Ship.**

Kit Includes:

- 2 Ice Packs
- 1 Colli-Pee Device
- 1 Collection Tube & Label
- 1 Patient Information Form
- 1 Bio-Hazard Bag

Thanks for choosing the ExoDx™ Prostate Test. Simply follow the step-by-step instructions in this guide to collect your sample right in the privacy of your own home. It's fast and easy!

Important: Please submit your sample as soon as possible, or as instructed by your health care provider.

View our ***Getting Started with the ExoDx Prostate Test*** video and follow along with written instructions.

www.exosomedx.com/how-to-order

Get started →

Day 1: Prepare

- 1 Place the ice packs—*found in the Styrofoam box*—in your freezer for at least 24 hours.
- 2 Complete the *Patient Information Form* provided in the kit.

Important: Add the **Required** Collection Date once the sample is collected

- 3 **Stop here.** Wait until ice packs are frozen solid for a minimum of 24 hours before completing Day 2 steps.

Important: Plan accordingly, samples should only be shipped Monday-Thursday.

Day 2: Collect


1 of 3

- 4 Only one label is needed from the bottom of the *Patient Information Form* for the collection tube.

Open the green package labeled **Colli-Pee**.


Write your full name, date of birth, and collection date on the label provided and attach it to the collection tube.

Unscrew the green cap on the collection tube, but keep it close by for when you are done. Securely attach the collection tube to the collection funnel.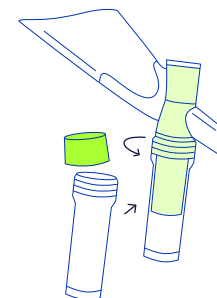

5 Urinate through the collection funnel.

Place penis into the collection funnel. Aim the smaller end of the collection funnel so that urine flowing through the funnel will go into the toilet. **Start and finish** urine flow into the funnel—excess urine will flow out the opposite side.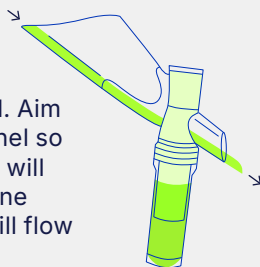

Important: This should be a 'first catch' sample, the first 15 mLs of the stream. Please allow collection tube to fill. The sample should be collected at least one hour from your last urination.

6 Detach the collection funnel, and securely click the collection tube closed.

Once urination is complete, unscrew and discard the funnel, and replace the green cap on the tube. The cap will click in place when secure. **Do not put the collection tube down before screwing the green cap on securely.**


Important: Immediately proceed to **Step 7**.

Before packing your sample, review shipping options on the following page.

7 Prepare sample for shipping.

- ☐ Open the provided bio-hazard bag and place the collection tube inside.
- ☐ Ensure the collection date is on both the Patient Information Form **AND** the collection tube label.
- ☐ Remove both frozen ice packs from your freezer, and **place them horizontally in the Styrofoam box.**
- ☐ If ice packs have not been frozen for a minimum of 24 hours, sample may be refrigerated until ice packs are ready for shipping.
- ☐ Place the collection tube between the two frozen ice packs.
- ☐ Place the Styrofoam lid on the styrofoam box. Place the completed *Patient Information Form* inside the box. Close the cardboard box and seal it with the self-adhesive tape strip.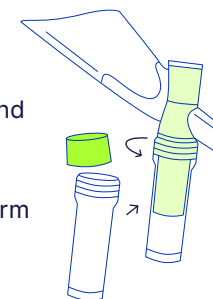

Ship

Choose Your Shipping Option
Ship Sample Monday-Thursday

Pickup From Home

Option 1: Contact FedEx at 1-800-463-3339 (1-800-GoFedEx) to schedule a pickup for your prepaid Priority Express Overnight box.

Option 2: Contact Client Services at 844-396-7663 to schedule a pickup.

Self Drop Off

Option 3: Drop off your sealed box at a local FedEx Office Shipping Center location. Must offer Express Overnight shipping.

Important to Know

- Please collect samples Monday-Thursday. Appropriately label the the Patient Information Form and collection tube with the collection date.
- Samples are shipped Priority Express Overnight with FedEx, but due to unforeseen circumstances, shipments could be delayed. Delayed samples may need to be resubmitted.
- The shipping label to return your sample will be attached to the outside of the blue box. The extra label is located on the back cover of these instructions, so you can track your sample to the lab.

What's Next?

Patient results will be delivered to your physician within 3-5 business days after receipt of sample.

Please contact Client Services at 844-396-7663 with any questions on the test, insurance coverage and the ExoCARES program.

The At-Home Kit is provided solely for use with the ExoDx Prostate Test. This test was developed and its performance characteristics determined by Exosome Diagnostics, Inc. This test is used for clinical purposes. It has not been cleared or approved by the U.S. Food and Drug Administration (FDA). The FDA has determined that such clearance is not necessary. Exosome Diagnostics, Inc. is certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA) as qualified to perform high-complexity testing.

La prueba de próstata ExoDx™

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This page is intentionally left blank.

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©2025 Exosome Diagnostics, Inc.
Exosome Diagnostics and ExoDx are trademarks of Exosome Diagnostics, Inc.